**▲ WARNING / AVERTISSEMENT / ADVERTENCIA :**

U.S. Federal Law restricts this device to sale by or on the order of a physician. Read all warnings and instructions before use. Improper use can result in serious or fatal illness or injury. This product contains dry natural rubber.

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ILLUSTRATIONS

Before reading these Instruction For Use, open this page to expose the illustrations. When reading Section V on Set-Up, the illustrations on the left inside flap will match the illustrations in the text. These illustrations will be helpful to you as they are referenced throughout these Instructions For Use.

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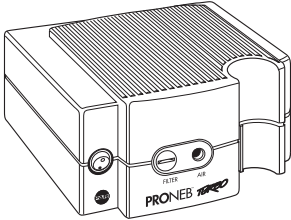
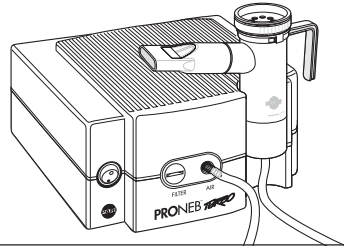
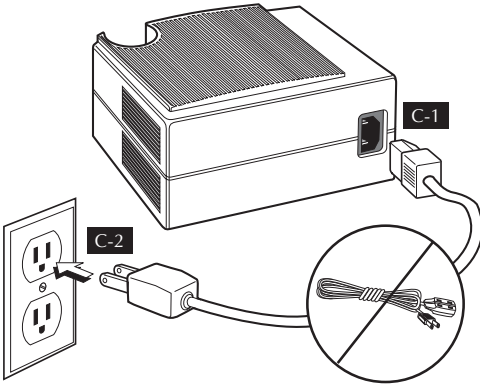
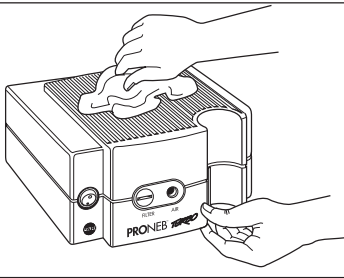
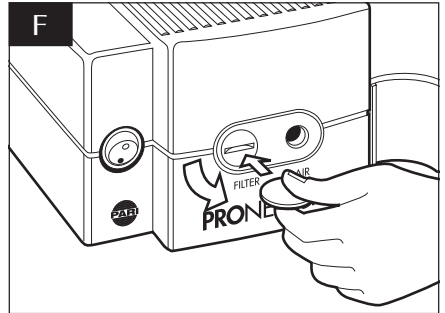
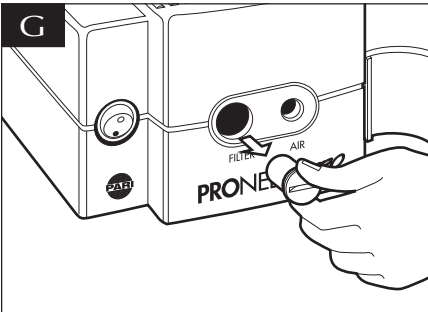
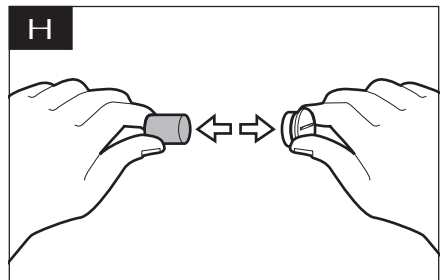
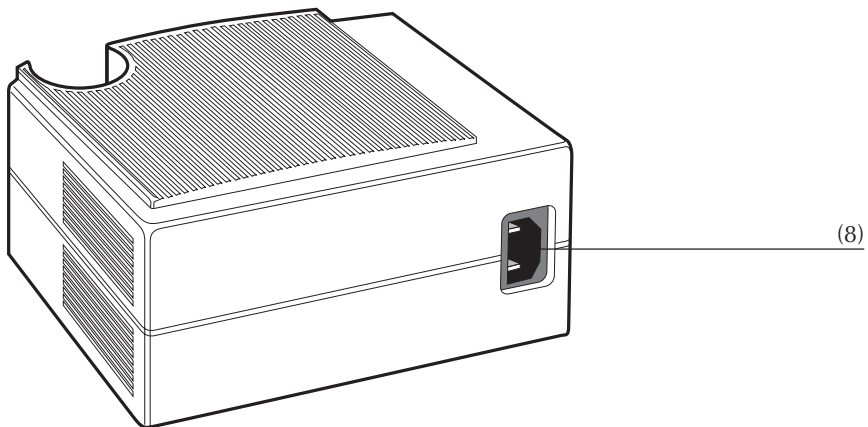
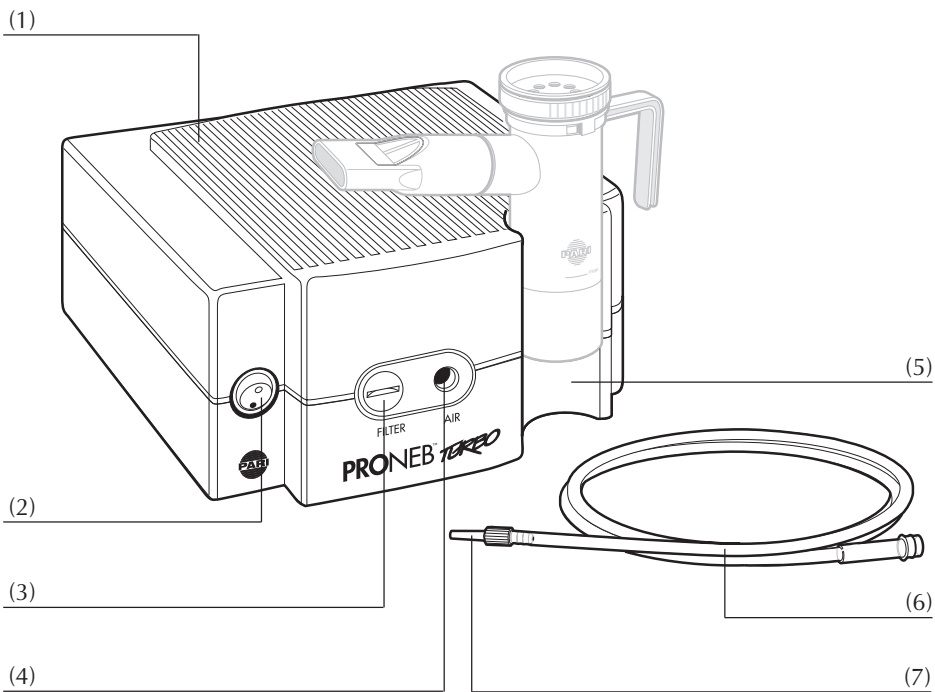
IMPORTANT SAFEGUARDS

⚠ WARNING

U.S. Federal Law restricts this device to sale by or on the order of a physician. Use only when prescribed by a physician and with the prescribed medication. Read all warnings and instructions before use. Improper use can result in serious or fatal illness or injury.

Use only with nebulizers and tubing authorized by PARI for this device. Other nebulizers and tubing may result in improper treatment.

When using electrical and medical products, especially when children are present, basic safety precautions should always be followed, including the following:

I. SAFETY PRECAUTIONS CONT.

⚠ WARNING

To reduce the risk of serious or fatal injury from electrocution, fire or burns and to reduce the risk of damage to the unit and malfunction:

1. Before use, check the specification plate on the bottom of the PRONEB TURBO compressor to ensure that the voltage and current indicated on the unit correspond to the voltage and current available.
2. Never operate this product if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or if it has been exposed to any liquids inside the case. Return the product to an authorized PARI Service Center for examination and repair.
3. Keep all electrical cords away from heated surfaces. Do not overload wall outlets or extension cords.
4. Never block the air openings of this product or place it on a soft surface, such as a bed or couch, where the air openings may be blocked. Keep the air openings free of lint, hair, and similar substances. Blockage could cause excessive heat build-up. A product should never be left unattended when plugged in.
5. Do not operate outdoors.
6. Do not operate where oxygen is being administered in a closed environment such as an oxygen tent.
7. Never drop or insert any object into any openings.
8. Always unplug this product immediately after using.
9. Be sure the PRONEB TURBO is unplugged prior to cleaning.
10. Do not open or disassemble the PRONEB TURBO Compressor. Refer all servicing to an authorized PARI Service Center.

⚠ DANGER

To reduce the risk of serious or fatal injury from electrocution:

1. Do not use or store product where it can fall or be pulled into a bath tub, sink, water, or other liquid. Do not place in or drop into water or other liquid. Do not use while bathing.
2. Do not reach for a product that has fallen into water or other liquid. Unplug immediately.

⚠ WARNING

To reduce the risk of serious or fatal injury and improper treatment:

1. Use the product only for its intended use as described in these Instructions For Use. Use only when prescribed by a physician and with the prescribed medication.
2. Do not use nebulizers, tubing or other attachments not recommended by the manufacturer.
3. Close supervision is necessary when this product is used by or near the physically or mentally impaired or children. Never use while sleeping or drowsy.

SAVE THESE INSTRUCTIONS**II. SPECIFICATIONS****COMPRESSOR SPECIFICATIONS:**

Weight:	4.2 lb. (1.9 Kg)
Dimensions:	6.5 x 3.7 x 5.5 in. (16.5 x 9.5 x 14.0 cm)
Electrical Requirements:	120V AC, 60Hz, 1.3 A or 220-240V, 50/60Hz
Power Wattage:	65 W @ 16 psi
Ambient Temperature Range:	10° C to 40° C, (50° F to 104° F)
Transport/Storage Range:	
Ambient temperature:	-40° C to 70° C, (-40° F to 158° F)
Relative Humidity:	10% to 100%, including condensation
Atmospheric Pressure:	500 hpa to 1060 hpa

EXPLANATION OF SYMBOLS:

ON/OFF

Type B

Insulation Class II

III. INTRODUCTION

These Instructions For Use contain information on the PRONEB TURBO Compressor (PRONEB TURBO). Separate Instructions For Use are included with the enclosed PARI Nebulizer.

Before using your PRONEB TURBO, you should read and understand the entire Instructions For Use for both the PRONEB TURBO and PARI Nebulizer. Make special note of all danger, warning and caution statements. Save this guide for future reference.

IV. GETTING STARTED

1. Check the PRONEB TURBO packing box and make sure all components are included.

DESCRIPTION:	PART NO.
PRONEB TURBO Compressor	
Air Filter (in compressor)	41E4851
PRONEB TURBO Instructions For Use	DS-38-810
Warranty Card (in nebulizer package)	DS-00-9999
Power Cord	41E8226
PARI Nebulizer Set with Instructions For Use	

If you did not receive all of the components, please contact the authorized dealer where you purchased your PRONEB TURBO.

2. Fill out the warranty card completely and mail.
3. It is recommended that the patient keep a back-up nebulizer on hand in case parts are lost or damaged.

V. SET-UP

If you have not done so already, please open the left flap of these Instructions For Use. The illustrations will come in handy.

1. Place the PRONEB TURBO on a stable, flat surface. Place the unit in an area that is clean and free of dust (**Fig. A**).
2. Connect one end of the power cord to the receptacle on the back of the PRONEB TURBO compressor (**Fig. C-1**). Connect the power cord plug to wall outlet (**Fig. C-2**).
3. Open the nebulizer package and read and follow the nebulizer's Instructions For Use.

Note: Your PRONEB TURBO compressor has a nebulizer holder on the front of the unit (**Fig. B**). This nebulizer holder allows the nebulizer to be placed in the front of the PRONEB TURBO with the tubing attached.

VI. MAINTENANCE OF THE PRONEB TURBO

TUBING MAINTENANCE

If there is any moisture or condensation in the tubing after the treatment, let the compressor run with only the tubing attached for 10 minutes. The warm air from the compressor will dry the moisture or condensation in the tubing. Alternatively, you can dry the tubing by removing it from the compressor and hanging it up with the “ends down” (**Fig. D**) to allow any moisture or condensation to drain out and evaporate. Use a damp cloth to wipe the exterior of the tubing.

CLEANING THE PRONEB TURBO HOUSING AND TUBING

Use a damp cloth to wipe the exterior of the PRONEB TURBO housing and tubing (**Fig. E**). **WARNING:** Never spray fluid into the ventilation slots on the PRONEB TURBO Compressor. Fluid could cause damage to the electrical and compressor parts and could lead to a malfunction. In the event that fluid enters the unit, return the product to an authorized PARI Service Center for examination and repair.

FILTER REPLACEMENT

Under normal conditions, the filter on the front panel of the PRONEB TURBO should be replaced after 500 hours of use, or sooner if it appears soiled (grey), clogged or wet. Check the condition of the filter after every 10-12 treatments. Do not attempt to wash and re-use filters. Do not use substitute filter material. Compressor damage can result. Using a small coin, turn the filter assembly (**Fig. F**) counterclockwise and remove from the compressor (**Fig. G**). The filter assembly is actually made up of 2 parts, the filter cover and the filter. Remove the filter from the cover by pulling apart (**Fig. H**). Insert the new filter with the closed end toward the filter cover. Insert the filter assembly into the filter chamber. Tighten the filter assembly by turning it clockwise using a coin. Do not over tighten. Replace with part #41E4851.

MAINTENANCE AND REPAIR

Never operate the PRONEB TURBO if it is damaged or improperly or incompletely assembled. Use only original PARI parts and attachments authorized by PARI Service Center for the PRONEB TURBO. Use of other parts and attachments may damage the unit or cause improper treatment. Do not open or disassemble the compressor unit at any time. Repairs, service and maintenance should be carried out only by authorized PARI Service Center.

Note: There are no user serviceable parts inside the PRONEB TURBO compressor case. Service by anyone other than authorized PARI Service Center voids warranty.

VII. REPAIR RETURN PROCEDURE

1. Contact PARI Patient Connection (Service Department) using the phone numbers listed in Section VIII (PRONEB TURBO Limited Warranty) of this booklet.
2. In writing, describe the malfunction or symptoms of the PRONEB TURBO. Include your full name, address and daytime telephone number.
3. Send written description and PRONEB TURBO in a protected box to the address provided by Patient Connection.

VIII. PRONEB TURBO LIMITED WARRANTY

PARI offers a limited five (5) year warranty on the PRONEB TURBO Compressor and its parts. PARI warrants the PRONEB TURBO compressor and its parts will be free from defects in workmanship and materials for a period of sixty (60) months from date of first retail purchase of the equipment. This warranty does not cover failures of the nebulizer, tubing, or accessories. This warranty does not extend to failures resulting from accident, misuse, abuse, alteration, use of unauthorized service, parts or cleaning solutions, or failure to comply with Instructions For Use.

If the warranted equipment should fail during the warranty period, PARI, at its option, will repair or replace the equipment or parts at issue, provided PARI determines to its satisfaction the claim is bona fide. It shall be the responsibility of the purchaser to pack and return the equipment in a manner to avoid shipping damage. The shipment must be prepaid to PARI Respiratory Equipment, Inc. This warranty covers the cost of labor incurred in the removal or replacement of the warranted component parts at authorized PARI service centers only. To make claim under this warranty, notify the authorized service center by calling PARI Customer Service at:

1.800.FAST.NEB (327.8632), Tel 1.804.639.7235, Fax 804.639.7242

THERE ARE NO OTHER EXPRESS WARRANTIES. ALL IMPLIED WARRANTIES ON THE PRONEB TURBO AND ITS PARTS, INCLUDING IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, ARE LIMITED TO SIXTY (60) MONTHS FROM THE DATE OF FIRST RETAIL PURCHASE. ALL EXPRESS OR IMPLIED WARRANTIES ON ANY OTHER PARTS, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, ARE DISCLAIMED. Some states do not allow time limitations on implied warranties, so the above limitations may not apply to your situation.

The sole and exclusive remedy under this warranty is to repair or replace the PRONEB TURBO and/or its parts at the option of PARI. IN NO EVENT SHALL PARI BE LIABLE FOR INCIDENTAL OR CONSEQUENTIAL DAMAGES TO THE PURCHASER OR ANY THIRD PARTY, INCLUDING WITHOUT LIMITATION, DAMAGES ARISING FROM PERSONAL INJURIES, LOST PROFITS, LOSS OF BUSINESS OPPORTUNITY, LOSS OF PROPERTY, ECONOMIC LOSSES, OR STATUTORY OR EXEMPLARY DAMAGES, WHETHER IN NEGLIGENCE, WARRANTY, STRICT LIABILITY OR OTHERWISE. Some states do not allow the exclusion or limitation of incidental or consequential damages, so the above limitations or exclusions may not apply to you.

This warranty gives you specific rights. You may also have other legal rights which vary from state to state.

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